DANSK

SVENSKA

SUOMALAINEN

NORSK

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PREVENTIVE PROTECTORS

# PROTECTIONS MALLEOLES EPITHELIUMTACT 04

MALLEOLAR PROTECTORS / KNÖCHELSCHUTZ / PROTEZIONI PER MALLEOLI / PROTEÇÕES PARA OS MALLEÓLOS / ENKELBESCHERMERS / PROTECCIONES PARA MALLEÓLOS / BESKYTTELSE TIL ANKELKNUDER / FOTKNÖLSSKYDD / KEHRÄSLUUN SUOJAT / ANKELPLASTER / ΠΡΟΣΤΑΤΕΥΤΙΚΑ ΑΣΤΡΑΓΑΛΩΝ

## BESKYTTELSER TIL ANKELKNUDER EPITHELIUMTACT 04

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### EPITHELIUMTACT 04

#### INDICATIONS:

EPITHELIUMTACT 04 malleolar protectors are made of self-adhesive silicone gel, of 2 mm thickness only, with visco-elastic properties similar to those of the skin. Applied to the malleoli in direct contact with the skin, the protectors distribute pressures and absorb frictions by dissipating energy in the gel and thereby prevent any risk of rubbing. They also relieve pain, allowing wearers to continue their sports activity, while maintaining their performance. Contact with the skin is very comfortable and the protector peels off painlessly, even when used on hairy areas.

#### CONTRAINDICATIONS:

- Do not use on an unprotected wound.
- Individuals with arterial disease, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

#### INSTRUCTIONS FOR USE:

- EPITHELIUMTACT 04 protectors have been specially designed to fully cover sensitive zones (malleoli, scaphoids).
- Remove the clear protective film (do not try to detach the fabric).
- Apply the protector directly to the skin, over the area to be protected. It can be removed and repositioned as required.
- It is preferable to cover with a sock.
- Place the protector on the storage carton provided and place in the foil sachet until next use.

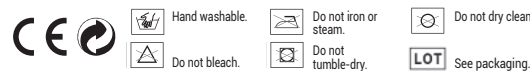
#### WASHING INSTRUCTIONS:

- The protectors can be washed with soap and water. Do not rub and leave to dry naturally.
- The protectors can be washed at least thirty times without affecting their properties.

**COMPOSITION:** 91% silicone, 9% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.

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