DANSK

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EPITHELIUMTACT 02

### INDICATIONS:

EPITHELIUMTACT 02 toenail protectors are made of ultra-resistant silicone gel, of 1 mm thickness only, with visco-elastic properties similar to those of the skin. The protectors thus distribute pressures and absorb micro-shocks by dissipating energy in the gel and thereby prevent the risk of bruised nails or blisters. They also relieve pain, allowing wearers to continue their sports activity, while maintaining their performance.

### CONTRAINDICATIONS:

- Do not use on an unprotected wound.
- Individuals with arterial disease, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

### INSTRUCTIONS FOR USE:

These protectors should be used **preventively**. They are also suitable for use once a bruised nail has formed.

The protectors must be used on clean, dry skin.

- For easy application, position the protector inside-out on the end of the toe, then gradually unroll it.
- Do not shorten the protector since this weakens it.
- Remove the protector at night to avoid any risk of maceration.
- Place the protector in the foil sachet provided until next use.

### WASHING INSTRUCTIONS:

The protectors can be washed with soap and water. Do not rub and leave to dry naturally.

**COMPOSITION:** 82% silicone, 18% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.

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