PORTUGUÊS

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ZA Champgrand - 26270 Lorient/Drôme - FRANCE

contact@epitact.com
www.epitact.com

ADJUSTABLE DIGITOPS with Epithelium 26®

INDICATIONS

The digitops are designed to prevent and relieve discomfort at the end and at the top of toes : soft corns, corns under the nail, dorsal corns, ingrown toenails.

CONTRA-INDICATIONS

- Do not use on a skin lesion.
- For persons suffering from arthritis, diabetis, polyneuritis, neuropathy of the lower limbs or varicose ulcers : keep a close check on the foot and in case of doubt, consult your doctor or your podiatrist.

PRECAUTIONS FOR USE

- cut them in the desired length.
- never cut them at the level of the seam.
- place the adjustable digitop on the corn.
- take it off each night to avoid maceration.

WHAT IT IS USED FOR

- The adjustable Digitop has no seam on the entire length.
- It has 2 wings that make it easy to place on the toe and can be cut to adapt the length of the Digitop to the toe that needs to be protected. Epithelium 26® is sandwiched between 2 layers of fabric at the "tip" of the toe protector for better wear resistance.

CLEANING

- Wash before first use. Machine–washable at 30°C or hand–washable without rubbing.
- Air dry (do not dry in dryer, on radiator or other source of artificial heat). Do not iron.

COMPOSITION: 75% silicone, 25% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993–1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2001.

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