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ENGLISH

DOUBLE PROTECTION INNER SOCKS with Epithelium 26®

Epithelium 26® is a silicone gel with mechanical properties close to those of the human foot pad.

INDICATIONS
The inner socks are for those who have very deformed feet, resulting in pressure and uncontrollable chafing in the majority of footwear.

CONTRA-INDICATIONS
• Do not use on a skin lesion.
• Persons with arteritis, diabetes, polyneuritis, peripheral neuropathy, or foot ulcers: consult your physician or chiropodist before application and monitor your foot during use.

TIPS FOR USE
• Wear them next to the skin for maximum comfort (except against open wounds).
• The printed part of the material, the longest side, must be in contact with the sole of the foot.
• The shortest length lies over your toes. If not, turn the sock over.
• For maximum comfort, wear sufficiently large shoes to avoid constricting your feet.
• Is applied like a normal sock or slipper. The inner socks are not intended to be worn on their own.

LASTING QUALITY
• Machine washable at 30°C or by hand (no twisting or rubbing).
• Air dry (do not dry in dryer, on radiator or other source of artificial heat). Do not iron.

COMPOSITION: 80% silicone, 20% polyamide/elastane fabric.
The biocompatibility of the materials has been assessed according to Standard ISO 10 993-1 for products which come into contact with the skin. If any reactions occur however you should stop using the support immediately and inform MILLET Innovation Company.

The MILLET Innovation Company cannot take any responsibility for use which does not comply with the manufacturer's recommendations. The reference language is French. Initial CE Mark: 2001.

	Machine washable at 30°C.	Do not iron or steam.	Do not dry clean.
	Do not bleach.	Do not tumble-dry.	See packaging.

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