PORTUGUÊS

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COMPOSITION: 84% silicone, 16% tissu polyamide/élasthanne.

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Ref. 10114-C_04/2017

ENGLISH

PLASTERS with Epithelium™ Activ INDICATIONS

- To prevent the formation of blisters in the back of the feet.
- To relieve pain from existing blisters.
- To prevent and alleviate the pain caused by pressure or friction.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- For persons suffering from arteritis, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers : keep a close check on the foot and in all cases seek the advice of your doctor or chiropodist.

PRECAUTIONS FOR USE

- On a clean and dry skin :
- Cut a piece of plaster for the area you want to protect.
- Take out the clear protective film (do not take out the cloth) and apply this module onto the skin. You can reuse the plaster, its natural self adhesiveness stays even after washing.
- The plaster is self adhesive by nature and can be removed without discomfort.
- It is recommended to apply one adhesive module on the plaster to maintain it (let 2cm or more on the sides and gently rub to avoid wrinkles).
- In case of use over existing blisters, pierce the blister, don't remove the dead skin and apply the plaster.

WHAT IT IS USED FOR

- The contact with the skin provides a refreshing sensation.
- The plaster can be removed without discomfort even on hairy areas.

RECOMMENDED CARE

- Hand washing (no twisting or rubbing).
- Air dry (do not dry in dryer, on radiator or other source of artificial heat). Do not iron.

COMPOSITION: 84% silicone, 16% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2006.

- | | | |
|----------------|-----------------------|-------------------|
| Hand washable. | Do not iron or steam. | Do not dry clean. |
| Do not bleach. | Do not tumble-dry. | See packaging. |

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