PORTUGUÊS

ESPAÑOL

DIGITUBES® com Epithelium™

INDICAÇÕES

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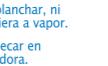
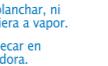
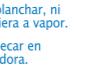
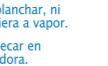
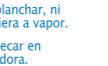
DIGITUBES® with Epithelium™

INDICATIONS

Protect toes or fingers, prevent formation of corns and blisters, and alleviate pain.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- Persons with arteritis, diabetes, polyneuritis, peripheral neuropathy, or foot ulcers: consult your physician or chiropodist before application and monitor your foot during use.

PRECAUTIONS FOR USE

- Use a good pair of scissors to cut the Digitube® to the required length.
- Remove the protective film covering the gel.
- Turn the Digitube® inside out and place it on the toe, making sure that the Epithelium™ gel is in contact with the area you wish to protect.

WHAT IT IS USED FOR

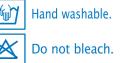
- Epithelium™ is a silicone gel with similar pressure distribution properties to those of human subcutaneous tissue. The presence of this substance in Digitubes® prevents corn formation and alleviates pain when calluses have already formed.
- Because they are not bulky, Digitubes® can be used on two consecutive toes at the same time.

CLEANING


- Hand-wash, without scrubbing.
- Leave to dry naturally. Do not iron.

COMPOSITION: 73% silicone, 27% polyamide/elastane fabric.

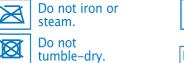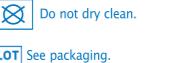
The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2001.

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