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## MICROFLUX™ DEVICES FOR SACRUM, ISCHIA, ELBOWS INDICATIONS

Aids in the prevention of bedsores on the sacrum, ischia, trochanter, elbows, knees, etc.

### INSTRUCTIONS FOR USE

- Apply to clean, healthy, dry skin.
- Carefully remove the protective film covering the adhesive surface (silicone gel).
- Hold the device by both sticky ends and position it on the affected area.

For heels and ankle bones, use the **MICROFLUX™ DEVICES FOR HEELS AND ANKLE BONES.**

### LIFESPAN

Several weeks under normal conditions of use.

### CARE

- Machine wash (40°C) in the laundry mesh bag provided or hand wash with soap and water (no rubbing)
- Allow to dry naturally. Do not iron.

### CONTRAINDICATIONS

- Do not apply to an open wound.
- In some instances, an allergic reaction (skin rash) may occur. In such a case, immediately discontinue use of the product and notify Millet Innovation SA.

MICROFLUX™ devices are to be used in combination with the standard preventive methods recommended by the National Agency for Accreditation and Evaluation in Healthcare (ANAES). As with any preventive aid, use of the devices cannot guarantee total protection and does not in any way forego regular monitoring of the skin condition of patients wearing them. Millet Innovation cannot be held responsible for the occurrence of bedsores during use of its devices.

MILLET will not incur liability for usage which does not comply with the manufacturer's recommendations.

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