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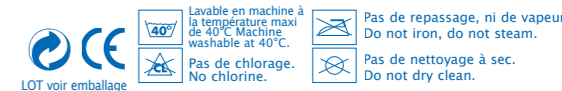
Ref. 10446-A

MICROFLUX™ DEVICES FOR HEELS AND ANKLE BONES

INDICATIONS

Aids in the prevention of bedsores on heels and ankle bones.

INSTRUCTIONS FOR USE

- Apply to clean, healthy, dry skin.
- Use the device on either the right foot or the left foot.
- Position the device making sure that the gel fully covers the heel and the ankle bones.
- Fasten the device on the top of the foot using self-adhesive strips.

For other susceptible areas, use **MICROFLUX™ DEVICES FOR SACRUM, ISCHIA AND ELBOWS**.

LIFESPAN

Several weeks under normal conditions of use.

CARE

- Machine wash (40°C) in the laundry mesh bag provided or hand wash with soap and water (no rubbing)
- Allow to dry naturally. Do not iron.

CONTRAINDICATIONS

- Do not apply to an open sore.
- In some instances, an allergic reaction (skin rash) may occur. In such a case, immediately discontinue use of the product and notify Millet Innovation SA.

MICROFLUX™ devices are to be used in combination with the standard preventive methods recommended by the National Agency for Accreditation and Evaluation in Healthcare (ANAES). As with any preventive aid, use of the devices cannot guarantee total protection and does not in any way forego regular monitoring of the skin condition of patients wearing them. Millet Innovation cannot be held responsible for the occurrence of bedsores during use of its devices.

MILLET will not incur liability for usage which does not comply with the manufacturer's recommendations.

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