## PORUGUÊS

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## ENGLISH

### ANTI-CRACK HEELS PROTECTIONS with Epithelium™

#### INDICATIONS

Effectively protects and relieves cracked, fissured, painful heels and improves their aesthetic appearance.

#### PROPERTIES

The protections distribute pressure and ensure occlusivity thanks to the Epithelium™ gel: they relieve pain when walking and significantly improve the appearance of heels in just 3 weeks.

- Worn in the day: they provide relief when walking and effective moisturization
- Worn at night: they provide effective moisturization

#### INSTRUCTIONS FOR USE

For cracked and painful heels, it is advisable to wear the protections during the day. If heels are cracked and unsightly, but not painful, the protections can be worn only at night.

- Apply the protections to clean, dry skin.
- Slide the forefront into the 2 device bands.
- Place the top band around the ankle.
- Adjust the part coated with the gel around the heel.
- Cover the protection with stockings, tights or socks if they are worn in the day.
- Repeat as indicated with the 2nd protection on the other foot.

It is advisable to apply the protections daily to enable the skin to renew itself, therefore providing significant improvement in the skin's appearance (disappearance of cracks, fissures and other superficial skin lesions).

#### CARE

- Machine-wash at 40°C in the washing net provided or wash by hand (without rubbing).
- Leave to air-dry. Do not iron.

#### CONTRAINDICATIONS

- The product has been tested according to ISO standard 10993-1. However, if an allergic reaction is observed, stop using the product immediately and contact Millet Innovation.
- Do not use on open wounds.
- Persons suffering from arteritis, diabetes, polyneuropathy, neuropathy of the lower limbs or varicose ulcers: never use the product on a wound, monitor the feet during use and ask your doctor or chiropodist for advice.

MILLET Innovation shall not be liable for any damages caused by use not in keeping with the manufacturer's recommendations.

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