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PLANTAR TIP PROTECTIONS with Epithelium 26®

Epithelium 26® is a patented silicone gel with mechanical properties close to those of the human footpad.

INDICATIONS

Thin or hard footpad due to age, calluses, rheumatism, prolonged standing, sport activities which are hard on the feet...

TIPS FOR USE

- Wear them directly on the skin (except on an open wound) .
- Slide your feet in the Plantar protections:
- the printed part of the fabric needs to be in contact with the ball of the foot. Do not turn inside out.
- The seam outside the foot shall slightly be higher on the top of the foot (see instructions).
- Then slip on socks, stockings or tights to keep them in place.

LASTING QUALITY

- Machine washable at 40°C in the mesh laundry bag or by hand (no twisting or rubbing).
- Air dry. Do not iron.

CONTRA-INDICATIONS

- This product has been put through all tests as prescribed by the ISO 10993 standard. In the unlikely event of an allergic reaction, discontinue use and notify MILLET Innovation SA.
- For persons suffering from arteritis, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: keep a close check on the foot and in case of doubt, consult your doctor or your podiatrist.

MILLET Innovation SA cannot be held responsible for use that does not conform with the manufacturer's recommendations.

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