PORTUGUÊS

CORRETOR com Epithelium™

INDICAÇÕES

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SPREADERS with Epithelium™

INDICATIONS

The Epithelium™ spreader limits deviation of the big toe (Hallux Valgus) by improving its alignment.

CONTRA-INDICATIONS

- Not for use on unprotected wounds.
- Special warning for persons with arteritis, diabetes, polyneuritis, lower-limb neuropathy or varicose ulcers: never use the product directly on a wound or perforating sore, watch foot throughout use and, in all cases, ask the opinion of your physician or podiatrist.

PRECAUTIONS FOR USE

- Put two half-spreaders together.
- Insert the resulting spreader between the 1st and 2nd toe: the widest part should be located under the foot.
- The half-spreaders alone is indicated for metatarsal corns.

WHAT IT IS USED FOR

- The Spreader helps align the big toe by preventing it from overlapping with neighbouring toes. EPITHELIUM™ is a silicone gel with mechanical properties close to those of some natural human tissue.

LASTING QUALITY

- It is washable (with soap and water) and re-usable.
- Lasts approximately 2 months (when directions for use are followed).

COMPOSITION: 100% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2004.

- Hand washable.
- Do not iron or steam.
- Do not dry clean.
- Do not bleach.
- Do not tumble-dry.
- See packaging.

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