PORTUGUÊS

ESPAÑOL

DANSK

SVENSKA

SUOMALAINEN

FRANÇAIS

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## VERRUCAS PROTECTIONS with Activ Epithelium™

### INDICATIONS

To protect the skin and provide relief from pain due to warts on the soles of the feet.

### CONTRAINDICATIONS

- The protections should not be applied to a wound or mucous membranes.
- In the event of an allergic reaction, or substantial irritation, the treatment should be stopped.

### PRECAUTIONS FOR USE

- Protections should be applied to clean, dry skin.
- Remove a protection from the metallic envelope.
  - Remove the protective film covering the adhesive side.
  - Apply the adhesive surface to the skin, taking care that the wart is well covered.
  - Apply light pressure to the protection to ensure that it has adhered to the foot.
  - Gently cover with socks, stockings or tights.
  - Keep the protection in place during the day.
  - Remove at night, place on the tray provided for this purpose and return to the metallic envelope until the next day.
  - Each protection may be used for an average of 5 days.
- It is recommended that the protection is applied everyday for at least a month. It is sometimes necessary to continue the treatment for a longer period, especially for established warts.

### WHAT IT IS USED FOR

These protections are suitable for adults and children over the age of three; they offer a safe, painless way of treating warts. They are also particularly recommended for diabetics.

- The protections, with Epithelium™ Activ gel, provide pain relief during walking by evenly distributing pressure on the wart and the hard skin surrounding it.
- The gel's occlusive properties help to accelerate the wart's disappearance.

### CLEANING

Protections may be washed with soap and water. Do not rub, allow to air-dry.

**COMPOSITION:** 87% silicone, 13% polyamide/elastane tissue.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2006.

- |                |                       |                   |
|----------------|-----------------------|-------------------|
| Hand washable. | Do not iron or steam. | Do not dry clean. |
| Do not bleach. | Do not tumble-dry.    | See packaging.    |

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