DANSK

SVENSKA

SUOMALAINEN

NORSK

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**PREVENTIVE PROTECTORS**

Protection en silicone  
EPITHELIUMTACT  
auto-adhésive  
EPITHELIUMTACT  
silicone protector  
self-adhesive

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### BLISTERS PROTECTORS EPITHELIUMTACT 01

#### FUNCTIONS:

EPITHELIUMTACT01 blisters protectors are made of self-adhesive silicone gel, of 1 mm thickness only, with visco-elastic properties similar to those of the skin. Applied to sensitive areas of the foot before sport, they distribute pressures and absorb frictions by dissipating energy in the gel and thereby prevent blisters forming. Contact with the skin is very comfortable and the protector peels off painlessly, even when used on hairy areas.

#### INSTRUCTIONS FOR USE:

- These protectors should be used preventively. They are also suitable for use once a blister has formed, as long as it has not burst.
- The protectors must be used on clean, dry skin.
- Take a protector out of its foil wrapper.
- Carefully peel off the protective film covering the adhesive side.
- Apply this adhesive side to the skin, being careful to fully cover the area to be protected.
- Press the protector gently to make sure it sticks properly.
- Cover the protector with holding adhesive to ensure impeccable hold (centre the protector, leaving a margin of 2 cm of adhesive around the edge).
- After use, remove the holding adhesive from the skin and carefully peel off the protector.
- Place the protector on the storage carton provided and place in the foil sachet until next use.

#### WASHING INSTRUCTIONS:

The protectors can be washed with soap and water 4 to 5 times on average without losing their adhesive properties. Do not rub and leave to dry naturally.

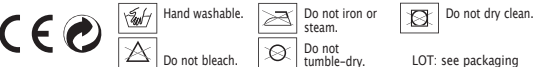
#### CONTRAINDICATIONS:

- The product has undergone all the regulatory tests required by ISO standard 10993-1 relative to products that come into contact with the skin. However, if any skin reactions are observed, stop using the product immediately and contact MILLET Innovation.
- Do not use on unprotected wound.
- Individuals with arterial disease, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

#### COMPOSITION:

- PROTECTOR: 84% silicone, 16% polyamide/elastane fabric.
- HOLDING ADHESIVE: 100% biocompatible.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use.

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