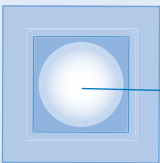
TECHNOLOGIE BREVETÉE PATENTED TECHNOLOGY

Ecarteur
Separator
Spreizer
Divaricatore
Distensor
Afstandhouder

Pastille en silicone
Silicone pad
Silikonscheibe
Piastrina in silicone
Pastilha de silicone
Kussentje in silicone

Petit manchon
Small sleeve
Kleine Manschette
Manicotto
Pequena meia
Klein omhulsel

Renfort thermoformable
Heat-mouldable insert
Thermoformbare Verstärkung
Rinforzo termoformabile
Reforço termomoldável
Thermovervormbare versterking

Cible blanche
White target
Weiße Marke
Bersaglio bianco
Guia branca
Witte marking

Grand bracelet
Wide wrap
Großes Band
Cinturino grande
Cinta grande
Grote band

PORTUGUÊS

ÓRTESE CORRETIVA DE USO NOTURNO

INDICAÇÕES

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NIGHT CORRECTIVE SPLINT

INDICATIONS

Regularly wearing the EPITACT® night corrective splint helps correct the deformity and limit the progression of your hallux valgus (bunion). It helps relieve night-time joint pain and eliminates pain caused by contact.

CONTRAINDICATIONS

- Big toe not mobile sideways.
- Do not use this product during the day.
- Do not use on a skin lesion.
- Do not use in the event of diabetes, arteritis, polyneuritis, peripheral neuropathy or foot ulcers.

PRECAUTIONS FOR USE

- First of all, make sure that the big toe can still move sideways when manipulated. Otherwise, do not use this product.
- The splint can be heat-moulded on either the right foot or the left foot.

– HEAT-MOULDING:

Follow the heat-moulding instructions supplied. The night corrective splint can be heat-moulded several times. If you think it's necessary, consult a podiatrist.

FITTING INSTRUCTIONS:

- Slip the forefoot into the wide wrap.
- Then slip the big toe into the small sleeve.
- Make sure the white target is positioned over your bunion (see diagram on the heat-moulding instructions supplied).

– USE CONDITIONS:

- It is recommended that the splint only be worn for 2 hours the first 3 times it is used.
- After this, you should wear the splint all night.

Joint pain can sometimes occur the first few times the splint is used. If this persists beyond 7 nights, consult your podiatrist.

- After wearing the splint for a month, and in the absence of pain, the splint may be heat-moulded a second time to increase the correction (see heat-moulding instructions supplied). In this case, the level of correction desired will be obtained without the separator, by pulling the big toe sideways (see diagram opposite).

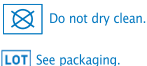
WASHING INSTRUCTIONS

- Machine wash at 30°C in the washing net provided or hand wash (without rubbing).
- Leave to dry naturally. Do not iron. Do not expose to heat (>50°C).

COMPOSITION: 52% heat-mouldable plastic, 28% polyamide/elastane fabric, 20% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2015.

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