
## TECNOLOGIE BREVETÉE PATENTED TECHNOLOGY

Gant en plastique  
Plastic glove  
Kunststoffhandschuh  
Guanto in plastica  
Luva de plástico  
Plastic handschoen

Renfort thermoformable  
Heat-mouldable insert  
Thermoformbare Verstärkung  
Rinforzo termoformabile  
Reforço termomoldável  
Thermovormbare versteviging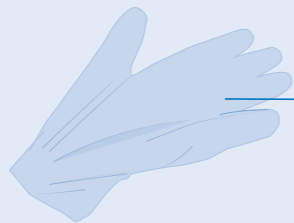

Sangle auto-agrippante  
Hook and loop strap  
Klettband  
Cinghia con chiusura a strappo  
Cinta autoadesiva  
Klittenbandriem

Produit issu de la recherche MILLET Innovation.  
Product designed by MILLET Innovation R&D.

[www.epitact.com](http://www.epitact.com)

## PORTUGUÊS

### ÓRTESE RÍGIDA NOTURNA

#### INDICAÇÕES

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## RIGID NIGHT SPLINT

### INDICATIONS

Regularly wearing the night splint helps relieve joint pain at the base of the thumb and limit the progression of basal thumb arthritis.

### FUNCTIONS

The EPITACT® rigid splint holds the thumb in a resting position, opening up the web space between the index finger and the thumb. The alignment of the bones limits the forces on the trapeziometacarpal joint and relieves pain. Custom-made since it is heat-mouldable, the EPITACT® splint is very comfortable and you will not even notice you are wearing it while you sleep.

### INSTRUCTIONS FOR USE

This product is a night use splint.

Two versions are available: right hand and left hand.

#### – HEAT-MOULDING:

Follow the heat-moulding instructions supplied.

The splint can be heat-moulded several times if necessary.

#### – FITTING INSTRUCTIONS:

- Slip the splint onto your hand by sliding your thumb into the thumb section.
- Insert the hook and loop strap through the loop at the base of the thumb and tighten the splint gently around the wrist.

### WASHING INSTRUCTIONS

- Machine wash at 40°C
- Do not use stain removers (acetone, solvents, etc.)
- Leave to dry naturally. Do not iron. Do not expose to heat (>50°C).

### CONTRAINDICATIONS

- The product has undergone all the regulatory tests required by ISO standard 10993-1. If, in spite of this precaution an incident occurs, discontinue use of the product immediately and inform MILLET Innovation.
- Do not use on an unprotected wound.
- In the event of severe oedema: monitor the hand during use. Remove the splint if it is too tight and impedes circulation. When in doubt seek advice from your GP or your pharmacist.
- Do not use near a flame or incandescent material.

**COMPOSITION:** 53% heat-mouldable plastic, 47% polyamide/elastane fabric. MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use.

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