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## INDICAÇÕES

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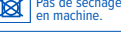
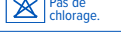
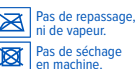
## CORRECTIVE ORTHOSIS FOR HALLUX VALGUS

## INDICATIONS

To correct the hallux valgus while relieving articular pain.

## CONTRAINDICATIONS

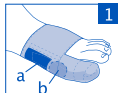
- Great toe that does not move.
- Do not use on a skin lesion.
- Persons suffering from arthritis, diabetes, neuropathy of the lower limbs or varicose ulcers: never use the product on a wound, monitor the feet during use and ask your doctor or chiropodist for advice.

## PRECAUTIONS FOR USE

- First check that the great toe can still move laterally when manipulated. If it cannot, do not use this device.
- Do not use this product overnight.
- The orthosis can be used on either the right or the left foot.
- Wash before first use.

## SET-UP :

- Insert the forefoot in the strap.
- Then put a hood on the great toe.
- Place the EPITHELIUM™ patch (a) behind the bunion (b). (See Diagram 1)

## - ADJUSTMENT:

- On the first day, do not try to straighten the great toe.
- Over 10 days, progressively straighten the great toe by pulling the device backwards. (See Diagram 2)

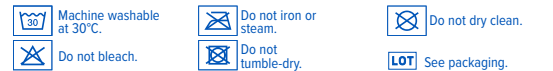
The device should be worn daily for maximum efficiency.

## CARE

- Washing after use at 30°C in the washing bag supplied or wash by hand (without rubbing).
- Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.
- Leave to air-dry. Do not iron.

**COMPOSITION:** 58 % silicone, 42 % polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (<0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2011.

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