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ENGLISH

MEDICAL DEVICE FOR FOR KELOID SCARS, Epithelium™

INDICATIONS
For improving the appearance of hypertrophic or keloid scars, whether recent or old, and preventing the formation of such scars, particularly after surgical intervention.

CONTRAINDICATIONS
• Do not use on a skin lesion.

PRECAUTIONS FOR USE
• Remove the patch from its packaging. Using scissors, cut out a piece slightly larger than the scar (2cm larger around the whole edge). Remove the protective film covering the adhesive side. Then apply the adhesive side to the scar.

In order to allow the skin to get used to it, the length of application time must be gradually increased as in the wearing times described below, until it is worn for 14 hours a day.
• Days 1 and 2: Wear for 4 hours / Day 3 and 4: Wear for 8 hours / Day 5: Wear for 10 hours / Day 6: Wear for 12 hours / Day 7: Wear for 14 hours

• After the 7th day, the Epithelium™ should ideally be worn for 14 hours per day and removed every evening.
• Following each use, the piece of dressing must be washed with soap and water.
The piece of dressing must be changed when it has deteriorated (after 2 weeks, on average).

In areas subject to a significant amount of friction, a hypafix type adhesive may be used to ensure that the Epithelium™ stays on the scar.
It must be applied very regularly, and can be used over a period of 2 to 4 months, depending on the improvements noted.

WHAT IT IS USED FOR
EPITACT™ Epithelium™ (silicone gel) dressing's effectiveness in improving the appearance of hypertrophic scars has been validated during a clinical study lasting 4 months. The results highlighted a significant decrease in redness, roughness and the area of the scar.

CLEANING
• Wash after use. Wash by hand, using soap and water.
• Discard and replace the device as soon as cracks in the gel or fabric damages occur.
• Leave to dry naturally in the protective case provided for this purpose. Do not iron.

COMPOSITION: 89% silicone, 19% polyamide/elastane fabric.
In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.
Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2009.

- | | | |
|---|---|--|
|  Hand washable. |  Do not iron or steam. |  Do not dry clean. |
|  Do not bleach. |  Do not tumble-dry. |  LOT See packaging. |

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