ÓRTESE CORRECTIVA DUPLA DO HALLUX VALGUS

INDICAÇÕES

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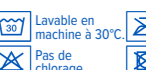
DOUBLE PROTECTION HALLUX VALGUS CORRECTIVE ORTHOSIS

INDICATIONS

To correct hallux valgus while relieving joint and plantar pain.

CONTRA-INDICATIONS

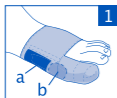
- Big toe that does not move sideways.
- Do not use on unprotected wound.
- Persons suffering from arthritis, diabetes, polynuropathy, lower limb neuropathy, varicose ulcers or severe oedema: never use the product on a wound, monitor the condition of your foot during use, and in all cases seek advice from your doctor or chiropodist.

PRECAUTIONS FOR USE

- This device should not be used when the big toe does not have sideways movement.
- Do not use this product during the night.
- There are two versions: left foot and right foot. The black part of the device should be in contact with the insole.
- Wash before first use.

- PLACEMENT:

- Insert the forefoot in the compression band.
- Cover the big toe with the hood.
- Position the elastic loop on the 2nd or 3rd toe (depending on the alignment of the plantar protective device).
- Place the Epithelium™ patch (a) behind the bunion (b). (See Diagram 1)

- ADJUSTMENT:

- On the first day, do not try to straighten the big toe.
- Over 10 days, progressively straighten the big toe by pulling the device backwards. (See Diagram 2)

The device should be worn daily for maximum efficiency.

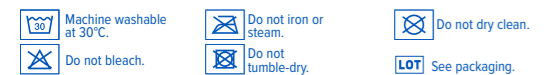
CLEANING

- Washing after use at 30°C in the washing bag supplied or wash by hand (without rubbing).
- Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.
- Leave to air-dry. Do not iron.

COMPOSITION: 56% polyamide/elastane fabric, 44% silicone.

In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2012.

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