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Ref.: 10628-H_06/2023

TOE TIP PROTECTIONS with Epithelium 26®

INDICATIONS

To protect and to alleviate pain due to friction and overpressure on top of the toes (corns, hammer, claw toes...)

CONTRA-INDICATIONS

- Do not use on a skin lesion.
- Persons with arthritis, diabetes, polyneuritis, peripheral neuropathy, or foot ulcers: consult your physician or chiropodist before application and monitor your foot during use.

PRECAUTIONS FOR USE

- Wash before first use.
- Wear them directly on the skin (except on an open wound) .
- Slide your feet in the Toe tip protections:
 - Position the lunula-shaped protection for toes on top of the toes.
 - The seam outside the foot shall slightly be higher on the top of the foot.
- Then slip on socks, stockings or tights to keep them in place.

WHAT IT IS USED FOR

- To protect the top of the toes from friction and overpressure (corns, hammer-toes...), these toe tip protections are made up of 1 mm lunula-shaped protection with Epithelium 26® and covered with fabric.
- The fabric has been selected for optimal comfort (softness, perspiration absorption) and an excellent upholding (elasticity).

CARE

- Washing after use at 30°C in the washing bag supplied or wash by hand (without rubbing).
- Discard and replace the device as soon as fabric or seam damages occur.
- Air dry. Do not iron.

COMPOSITION: 50 % silicone, 50 % polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2003.

Machine washable at 30°C.	Do not iron or steam.	Do not dry clean.
Do not bleach.	Do not tumble-dry.	See packaging.

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