## CORRETOR com EPITHELIUM™

## INDICAÇÕES

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## SPREADERS with EPITHELIUM™

### INDICATIONS

The EPITHELIUM™ spreader limits deviation of the big toe (Hallux Valgus) by improving its alignment.

### CONTRAINDICATIONS

- Do not use on a skin lesion.
- Special warning for persons with arteritis, diabetes, lower-limb neuropathy or varicose ulcers: never use the product directly on a wound or perforating sore, watch foot throughout use and, in all cases, ask the opinion of your physician or podiatrist.

### PRECAUTIONS FOR USE

- Wash before first use.
- Put two half-spreaders together.
- Insert the resulting spreader between the 1st and 2nd toe: the widest part should be located under the foot.
- The half-spreaders alone are indicated for metatarsal corns.

### WHAT IT IS USED FOR

- The Spreader helps align the big toe by preventing it from overlapping with neighbouring toes. EPITHELIUM™ is a silicone gel with mechanical properties close to those of some natural human tissue.

### CARE

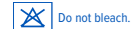
- Wash after use. Protections may be washed with soap and water.
- Discard and replace the device as soon as cracks in the gel appear.
- Lasts approximately 2 months (when directions for use are followed).

### COMPOSITION: 100% silicone.

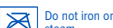
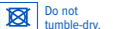
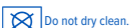
In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2004.

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