ALMOFADAS DISCRETAS com Epithelium 26®

INDICAÇÕES

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ENGLISH

DISCREET CUSHIONS with Epithelium 26®

INDICATIONS

Thin or hard plantar tissue due to ageing, calluses, extended standing.

CONTRA-INDICATIONS

- Do not use on a skin lesion.
- For persons suffering from arteritis ,diabetes, neuropathy of the lower limbs or varicose ulcers: keep a close check on the foot and in case of doubt, consult your doctor or your chiropodist.

PRECAUTIONS FOR USE

- Wash before first use.
- Place them directly on the skin for optimal comfort.
- To determine the correct side, right from left, place the cushions in front of you, making sure the printing is right, side up. A space at the base forming a triangle will confirm that both cushions are positioned correctly and ready to be inserted in your shoes. Otherwise, reverse them.
- Remove the protective film from the double-sided tape found on the back of the cushions.
- Position the cushions in your shoes, in the widest area. The cushions will be slightly slanting, as shown here.

CLEANING

- Wash after use. You may machine-wash your cushions (30°C) and change the adhesive. If you do so, let the cushions air-dry and change the adhesive strip.
- Discard and replace the device as soon as fabric or seam damages occur.

COMPOSITION: 81% silicone, 19% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2005.

Machine washable at 30°C.	Do not iron or steam.	Do not dry clean.
Do not bleach.	Do not tumble-dry.	See packaging.

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