Ref.: 19377_F_06/2023

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RIGID BUNION CORRECTOR HALLUX VALGUS BUNION

INDICATIONS

Regularly wearing the EPITACT® night corrective splint helps correct the deformity of your hallux valgus (bunion). It helps relieve night-time joint pain and eliminates pain caused by contact.

CONTRAINDICATIONS

- Big toe not mobile sideways.
- Do not use this product during the day.
- Do not use on a skin lesion.
- Do not use in the event of diabetes, arteritis, polyneuritis, peripheral neuropathy or foot ulcers.

PRECAUTIONS FOR USE

- First of all, make sure that the big toe can still move sideways when manipulated. Otherwise, do not use this product.
- The splint can be heat-moulded on either the right foot or the left foot.

- HEAT-MOULDING:

Follow the heat-moulding instructions supplied. The night corrective splint can be heat-moulded several times. If you think it's necessary, consult a podiatrist.

FITTING INSTRUCTIONS:

- Slip the forefoot into the wide wrap.
- Then slip the big toe into the small sleeve.
- Make sure the white target is positioned over your bunion (see diagram on the heat-moulding instructions supplied).

- USE CONDITIONS:

- It is recommended that the splint only be worn for 2 hours the first 3 times it is used.
- After this, you should wear the splint all night.

Joint pain can sometimes occur the first few times the splint is used. If this persists beyond 7 nights, consult your podiatrist.

- After wearing the splint for a month, and in the absence of pain, the splint may be heat-moulded a second time to increase the correction (see heat-moulding instructions supplied). In this case, the level of correction desired will be obtained without the separator, by pulling the big toe sideways (see diagram opposite).

WASHING INSTRUCTIONS

- Wash after use. Machine wash at 30°C in the washing net provided or hand wash (without rubbing).
- Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.
- Leave to dry naturally. Do not iron. Do not expose to heat (>50°C).

COMPOSITION: 52% heat-mouldable plastic, 28% polyamide/elastane fabric, 20% silicone.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2015.

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