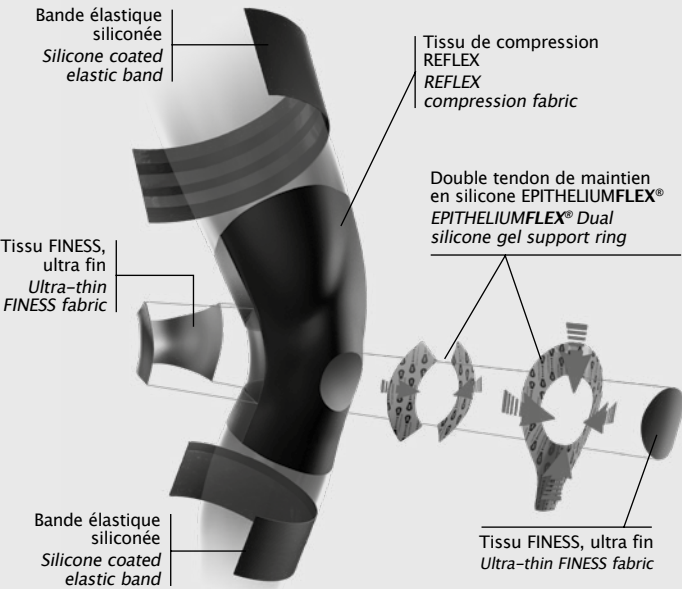
## PATENTED TECHNOLOGY

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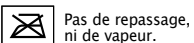
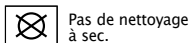
# PHYSIOOstrap™ Ski

EPITHELIUMFLEX® 03

**INDICATIONS:** Wearing the PHYSIOOstrap™ Ski helps stabilise the knee joint during board sports which involve repeated flexion and extension movements. It relieves kneecap disorders (patellofemoral pain syndrome, patellar instability, patellar tendinitis etc.), or pain resulting from the residual effects of previous injuries (sprain, ligament or meniscus lesion etc.)

**CONTRAINDICATIONS:** To ensure that blood circulation is not impeded, do not use if you have arteritis, severe varicose conditions, sequelae of phlebitis or oedema (lymphatic/venous) of the lower leg and/or the thigh. Do not use with compression stockings. Do not use on a skin lesion.

### PRECAUTIONS FOR USE:

• To avoid any irritation caused by the silicone elastic bands, the PHYSIOOstrap™ Ski should not be worn in a static position, particularly with the wearer lying down (sleep) or sitting (e.g. during bus or car journeys, lunch breaks etc.). This PHYSIOOstrap™ Ski should be put on right before starting board sports activities and worn only during exercise.

• If you experience pain, oedema, discomfort or any other abnormal reaction, stop using the product and consult a doctor.

**HOLD:** Do not apply moisturising products to the skin before using the PHYSIOOstrap™ Ski to ensure optimum hold. **IMPORTANT!** Place the PHYSIOOstrap™ Ski on the thigh without stretching it. To guarantee good hold, the product must be washed after each use. At the very least, wash the silicone elastic bands with soap and water (without scrubbing them) to remove any dirt.

**ERGONOMIC DESIGN:** The PHYSIOOstrap™ Ski is a concentrated technology weighing less than 70g, which has been specifically designed for board sports (skiing, snowboarding etc.) Its patented silicone patellar support ring, combined with REFLEX ultra-effective compression fabric, provides optimal kneecap support and knee stability. PHYSIOOstrap™ Ski is designed to fit either the right or the left knee. It does not slip and does not cause any discomfort at the back of the knee (popliteal fossa). Extra-thin, it can be worn under all types of clothing, even technical clothing (non-compressive).

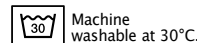
**WASHING INSTRUCTIONS:** Wash after use. Machine wash at 30°C or hand wash (without rubbing), leave to dry naturally. Do not iron. Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.

**COMPOSITION:** 70% polyamide/elastane fabric, 30% silicone.

In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

If you experience pain, oedema, discomfort or any other abnormal reaction, stop using the product and consult a doctor.

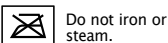
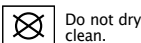
MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2016.

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