PATENTED TECHNOLOGY

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COMPOSITION: 71% tissu polyamide/élasthane, 29% silicone.

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ENGLISH

PHYSIOstrap™

INDICATIONS: Wearing the EPITHELIUMFLEX® PHYSIOstrap™ helps to relieve and protect your delicate or painful knee (osteoarthritis, patella diseases, fragile ligaments, etc.). It helps to bring a feeling of stability and to continue or return to everyday activities forgetting about it.

CONTRAINDICATIONS: To ensure that blood circulation is not impeded, do not use if you have arteritis, severe varicose conditions, sequelae of phlebitis or oedema (lymphatic/venous) of the lower leg and/or the thigh. Do not use with compression stockings. Do not use on a skin lesion.

PRECAUTIONS FOR USE:

- Wash the PHYSIOstrap™ before the first use.

- The PHYSIOstrap™ has been designed to be used dynamically: **DO NOT WEARING IT FOR LONG PERIODS OF TIME WHEN YOU ARE SEATED OR LYING DOWN.** Do not bend your leg to less than 90° when you are seated.

- If you experience pain, oedema, discomfort or any other abnormal reactions stop using the product and consult a doctor.

WEARING THE SUPPORT:

- In order to wear the knee brace optimally:
- Do not apply moisturising cosmetics before using the PHYSIOstrap™.
- To guarantee good hold, the product must be washed before each use. At the very least, wash the silicone elastic bands with soap and water (without scrubbing them) to remove any dirt.

ERGONOMICS: Extra thin and discrete, the EPITHELIUMFLEX® PHYSIOstrap™ is a concentrated technology weighing less than 40g, which has been specifically designed to be worn under all types of clothing (trousers, skirts, etc.). It does not slip and does not cause any discomfort at the back of the knee (popliteal fossa).

WASHING INSTRUCTIONS: Wash after use. Machine wash at 30°C or hand wash (without rubbing), leave to dry naturally. Do not iron. Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.

COMPOSITION: 71% polyamide/elastane tissue, 29% silicone.

In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

If you experience pain, oedema, discomfort or any other abnormal reaction, stop using the product and consult a doctor.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2016.

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