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**COMPOSITION** : 80% silicone, 20% tissu polyamide/elasthanne.

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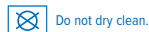
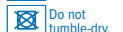
Do not tumble-dry.

See packaging.

## DOUBLE PROTECTION INNER SOCKS with Epithelium 26®

Epithelium 26® is a silicone gel with mechanical properties close to those of the human foot pad.

### INDICATIONS

The inner socks are for those who have very deformed feet, resulting in pressure and uncontrollable chafing in the majority of footwear.

### CONTRA-INDICATIONS

- Do not use on a skin lesion.
- Persons with arteritis, diabetes, peripheral neuropathy, or foot ulcers: consult your physician or chiropodist before application and monitor your foot during use.

### PRECAUTIONS FOR USE

- Wash before first use.
- Wear them next to the skin for maximum comfort (except against open wounds).
- The printed part of the material, the longest side, must be in contact with the sole of the foot.
- The shortest length lies over your toes. If not, turn the sock over.
- For maximum comfort, wear sufficiently large shoes to avoid constricting your feet.
- Is applied like a normal sock or slipper. The inner socks are not intended to be worn on their own.

### CLEANING

- Washing after use at 30°C in the washing bag supplied or wash by hand (without rubbing).
- Discard and replace the device as soon as fabric or seam damages occur.
- Leave to dry naturally. Do not iron.

**COMPOSITION**: 80% silicone, 20% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

The MILLET Innovation Company cannot take any responsibility for use which does not comply with the manufacturer's recommendations. The reference language is French. Initial CE Mark: 2001.

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