ÓRTESE RÍGIDA NOTURNA

INDICAÇÕES

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RIGID NIGHT SPLINT

INDICATIONS

Regularly wearing the night splint helps relieve joint pain at the base of the thumb and limit the progression of basal thumb arthritis.

CONTRAINDICATIONS

- Do not use on an unprotected wound.
- In the event of severe oedema: monitor the hand during use. Remove the splint if it is too tight and impedes circulation. When in doubt seek advice from your GP or your pharmacist.
- Do not use near a flame or incandescent material.

FUNCTIONS

The EPITACT® rigid splint holds the thumb in a resting position, opening up the web space between the index finger and the thumb. The alignment of the bones limits the forces on the trapeziometacarpal joint and relieves pain. Custom-made since it is heat-mouldable, the EPITACT® splint is very comfortable and you will not even notice you are wearing it while you sleep.

INSTRUCTIONS FOR USE

This product is a night use splint.

Two versions are available: right hand and left hand.

- HEAT-MOULDING:

Follow the heat-moulding instructions supplied.

The splint can be heat-moulded several times if necessary.

- FITTING INSTRUCTIONS:

- Slip the splint onto your hand by sliding your thumb into the thumb section.
- Insert the hook and loop strap through the loop at the base of the thumb and tighten the splint gently around the wrist.

WASHING INSTRUCTIONS

- Wash after use. Machine wash at 30°C in the washing net provided or hand wash (without rubbing).
- Discard and replace the device as soon as fabric or seam damages occur.
- Do not use stain removers (acetone, solvents, etc.)
- Leave to dry naturally. Do not iron. Do not expose to heat (>50°C).

COMPOSITION: 53% heat-mouldable plastic, 47% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2015.

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