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BLISTER PREVENTION PADS with active Epithelium™

INDICATIONS

To protect the skin, to prevent and provide relief from blisters, irritations, branderigheid door wrijving etc.

CONTRA-INDICATIONS

- Do not use on a skin lesion.
- In the event of an allergic reaction, or substantial irritation, the treatment should be stopped.

PRECAUTIONS FOR USE

These protection pads are to be used for the prevention of blisters. They are also ideal for blisters that have formed but not burst. protection pads should be used on clean, dry skin.

- Remove a pad from the metallic envelope.
- Remove the protective film covering the adhesive side.
- Apply the adhesive surface to the skin, taking care that the area to be protected is well covered.
- Apply light pressure to the pad to ensure that it has adhered.
- After use place the pad on the storage tray provided for this purpose and return to the metallic envelope until the next use.

WHAT IT IS USED FOR

- **Epithelium™ active is a self-adhesive gel that bonds with the skin and reduces friction at the skin's surface.** This provides protection for the skin and relief from blisters that have formed but not burst.
- Pleasant to wear, **these protective pads are repositionable and painless to remove**, even from areas with hair growth. They may be used on the feet, in all types of shoes (formal, sports etc.), and on any area of the body subject to rubbing (e.g. due to clothing or irritation caused by skin rubbing against skin).

These pads are particularly recommended for diabetics.

CLEANING

- Wash after use. Pads may be washed with soap and water. Do not rub, allow to air-dry.
- Discard and replace the device as soon as cracks in the gel or fabric damages occur.
- Leave to dry naturally. Do not iron.

COMPOSITION: 89% silicone, 11% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2006.

	Hand washable.		Do not iron or steam.		Do not dry clean.
	Do not bleach.		Do not tumble-dry.		See packaging.

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