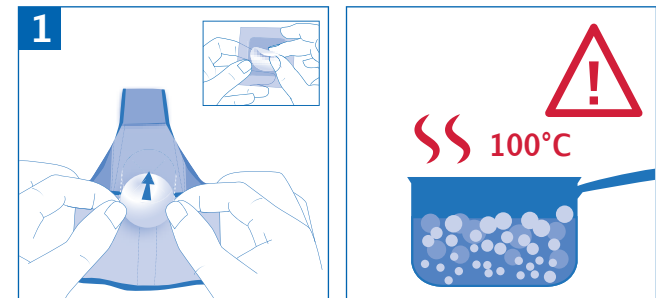
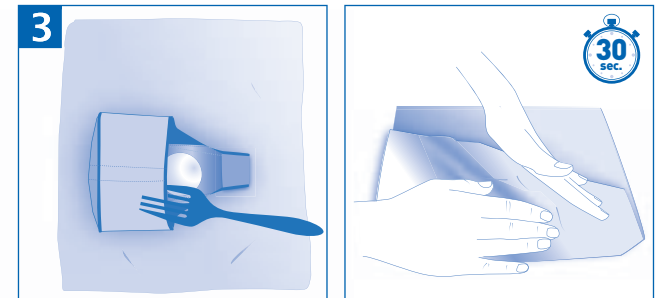
NOTICE DE THERMOFORMAGE

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

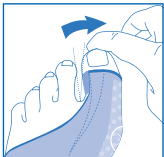
HEAT-MOULDING INSTRUCTIONS

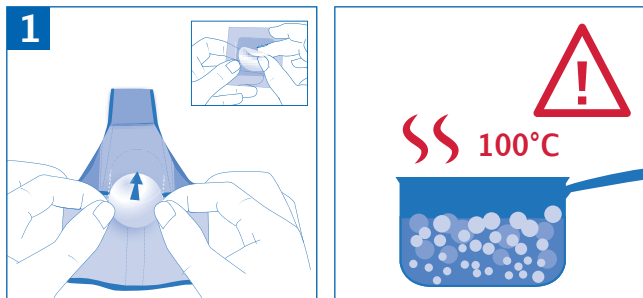
Before you start, it is essential to read all of the heat-moulding instructions to ensure that you have fully understood all the steps. We also recommend that you watch the video instructions presented on www.epitact.com

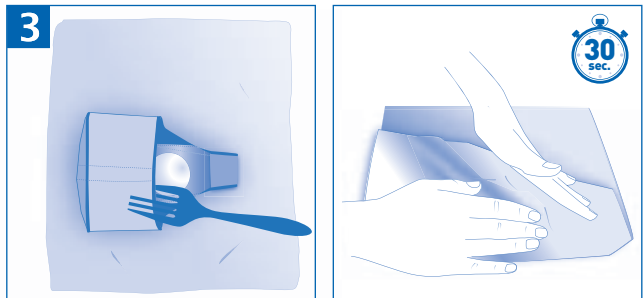Remove the silicone pad from the plastic and place the flat surface on the blue circle inside the splint. Boil 1 litre of water in a saucepan. **It is essential to wait until the water is boiling properly to ensure a temperature of 100°C.**

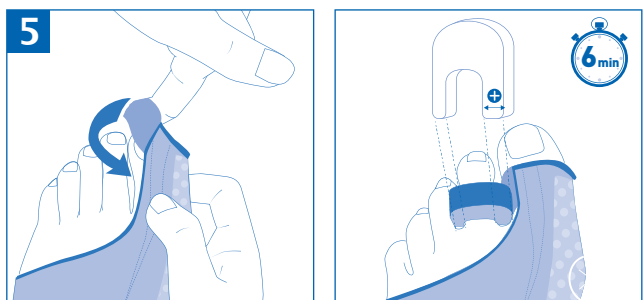Use the fork to remove the splint from the water and carefully soak up excess water in a dishcloth for 30 to 40 seconds.


Then slip the small sleeve around the toe. Realign the big toe using the separator. **The wide part of the separator must be positioned on your big toe side.** Leave in place for at least 6 minutes, remaining seated with your foot on the floor.


IMPORTANT INFORMATION:
It is essential to use the separator in order to prevent excessive correction. To limit the joint pain that can sometimes occur in the first few days, it is recommended that the splint only be worn for 2 hours the first 3 times it is used.

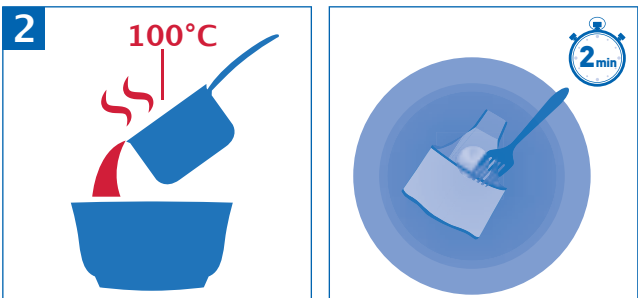
NEED FOR HEAT-MOULDING:
1 saucepan (to boil water at 100°C)
1 container (heat-resistant)
1 fork
1 dishcloth
1 watch
ESTIMATED TIME: 10 min.

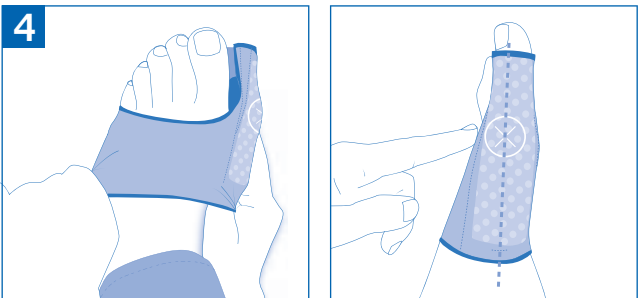Place the splint in a container and pour **1 litre of boiling water onto it (100°C)**. Use a fork to hold the splint under the water at the bottom of the container for 2 minutes to soften the heat-mouldable insert.

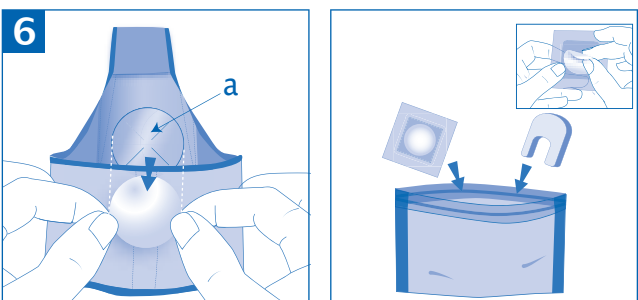
Hold the wide wrap open and slip on the splint, **positioning the white target over your "bunion"**. Check the correct lateral position of the insert.

Remove the separator and the foot splint. **Remove the silicone pad** and check that the hollow obtained is located at the level of the blue circle (a). Put the pad and the separator away in the silver bag for subsequent heat-moulding if required. Leave the splint to dry naturally.

CHECKING HEAT-MOULDING:
The hollow inside the splint must be located at the level of the circle (a). The presence of the fold is normal (b). The separation obtained (c) is deliberately moderate the first time heat-moulding is performed with the separator. If you are not satisfied, repeat the operations 1 to 6 (the splint can be heat-moulded several times).

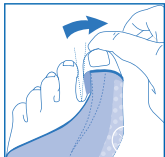After a month, the splint may be heat-moulded a second time to increase the correction. In this case, the level of correction desired will be obtained without the separator, by pulling the big toe sideways.

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